



## Description

**PULSE PACK** is a **BUILT IN BATTERY PACK** and fully Disposable single use instruments, designed for use in Orthopedic Surgical procedures, trauma and general wound care.

Pack includes :	Hand Piece	-	1 No
	Small Cone Tip	-	1 No
	Coaxial Tip	-	1 No

## Advantages

- No need to hang separate battery pack
- Very easy to disconnect the cable connected on the battery pack
- No risk of electric shock
- The separate battery pack has a potential for electric power leakage during disconnecting the electric cable after use.



30mm CONE TIP



COAXIAL CANAL TIP

Imported & Marketed by :

**BELL SURGICAL**

An ISO 9001 : 2008 | Certified Company

Unit No. 68, Part A, Municipal Industrial Estate,  
Gandhi Nagar, Worli, Mumbai - 400 018. INDIA  
Phones : +91 24953749 / +91 9820577304  
+91 9320229766 / 9870383683  
email: nitishbellare@bellsurgical.com



# PULSE PACK

# BS

## Instruction for Use

- Peel open the pack and remove all items
- Retract Locking Ring by Pressing up (A)
- Attach the required tip to the handle of the gun and push on until the connection is secure (When using the Intra Medullary Tip ensure the white cap is removed before use.)
- Lock the required tip in place by depressing the Locking Ring . Retract the Locking Ring for Tip Removal (A)
- Attach the irrigation line to the irrigation source (D)
- Attach the suction line to the suction carrier source(F)
- Remove the protector from the toggle Trigger (C)
- Press the TOP for High Speed
- Press the Bottom of the toggle trigger for low speed delivery
- Centre the Toggle Trigger for no fluid delivery.
- Press the Top position of Trigger at high Speed to start while the First time use.
- The device has a clamp on irrigation line which can be used in the event of leakage of irrigation fluid.
- The device has a clamp on the suction line which can be adjusted.

## Warnings

- Ensure that the device is disposed of safely and in accordance with all current national and international waste disposable directives.
- Store in cool and dry place.
- Single use only..... Do not Re-Sterilize.....

## Specification

- DC 10.5V Battery
- Tube Length 3 meters
- High Speed Flow Rate up to 1000ml / M
- Low Speed flow Rate 700ml / M
- Pressure up to 15 PSI

Imported & Marketed by :

**BELL SURGICAL**

An ISO 9001 : 2008 | Certified Company

Unit No. 68, Part A, Muncipal Industrial Estate,  
Gandhi Nagar, Worli, Mumbai - 400 018. INDIA  
Phones : +91 24953749 / +91 9820577304  
+91 9320229766 / 9870383683  
email:nitishbellare@bellsurgical.com



Attention  
See instruction  
for use

